


EsCem® RMGI Cementation Technical Guide



CEMENTATION GUIDE for all inlays, onlays, crowns, bridges. (metal, gold, PFM)




1. Clean, rinse and thoroughly dry the prepared tooth.

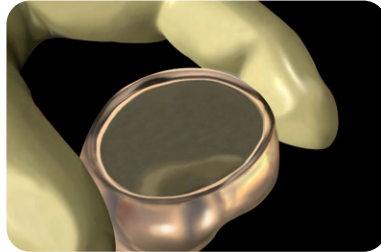

2. Prepare the restoration according to the manufacturer's instructions.




3-1. Place the eco tip and dispense EsCem® RMGI onto a clean mixing pad.

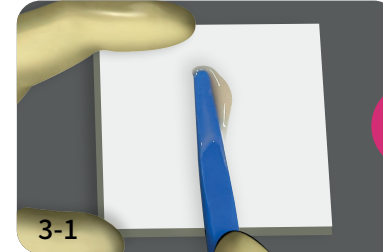


3-1. Use a mixing spatula to homogenously mix EsCem® RMGI and load the mixed cement into the restoration.




3-2. Place the mixing tip and apply EsCem® RMGI directly into the restoration.

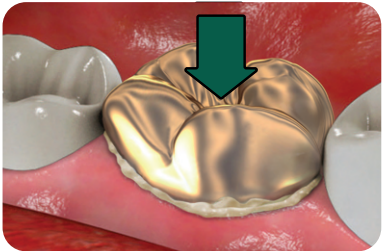


4. Seat immediately and maintain proper pressure.

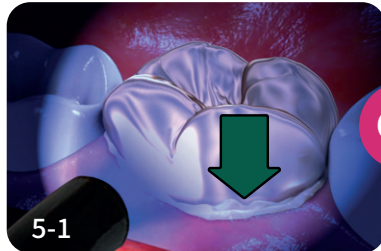


5-1. Tack cure 5 seconds using curing light to remove excess material.

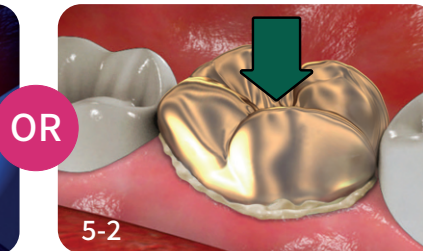

5-2. Keep proper pressure until it reaches gel paste for 90 seconds.

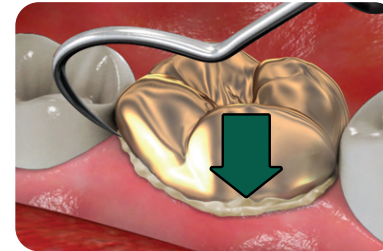

6. Remove excess EsCem® RMGI while maintaining proper pressure.

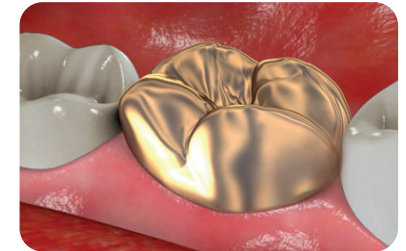

7. Let the material set for 4 minutes in case the light didn't pass through the crown.