

You're
the Artist



Dependable Bonding Performance



Light-cured 5th bonding agent,
which combined primer and adhesive
in a single bottle

Indication



- Direct composite restorations
- Indirect restorative cementation: metal, porcelain, composite
- Composite, porcelain, amalgam repairs

Advantages

- Superior bonding strength
- Wet bonding system
- Acetone based, 5th generation adhesive

With millions of bonded restorations being placed every year,
what we need is dependable and predictable results.

Physical Characteristics

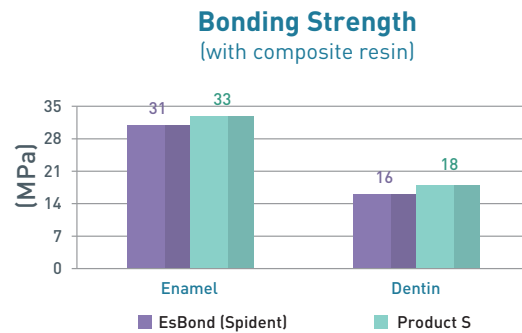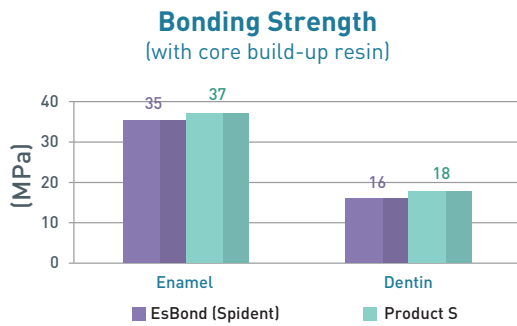


(For use with EsBond Activator)

Officially tested by Yonsei University College of Dentistry


Application



1. Apply EsBond

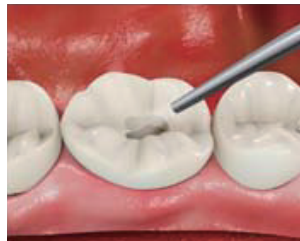

2. Air dry the cavity

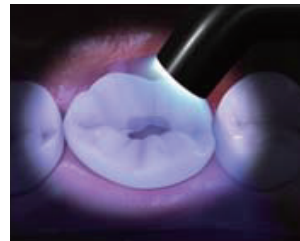

3. Light-Cure for 20 seconds



4. Place the EsFlow or EsCom100

Related products

EsBond Activator

EsBond Activator activates the dual-cure mechanism of EsBond bonding system.
When mixed, EsBond becomes dual cure and adhere securely to dual-cure restoration.



SPIDENT CO., LTD.

203&312, Korea Industrial Complex, 722, Gojan-dong, Namdong-gu, Incheon, Korea 405-821

Tel. 82-70-5057-0827 Fax. 82-32-821-0074 E-mail. synergy@spident.co.kr

www.spident.co.kr